

Create it ▲

TIP GUIDE

ULTRASONIC BONE
SURGERY



NSK

VARIOSURG TIP LINEUP


SURG	
BONE SURGERY	P05-P12
SCRAPER	P13-P14
EXTRACTION	P15-P16
SOCKET LIFT (CRESTAL APPROACH)	P17-P20
IMPLANT PREPARATION	P21-P24
SOCKET LIFT (CRESTAL APPROACH)	P25-P26*
SINUS LIFT	P27
SINUS MEMBRANE DETACHMENT	P28
PERIO	
SCALING	P29-P31
PERIO	P32
PERIO-CONTROL	P33-P34
ENDO	
ENDODONTICS	P35-P36
TIP KIT	P37-P40

*Pictures provided by Yasuhiro Kizu, D.D.S., Ph.D.
OMFC (ORAL & MAXILLOFACIAL CARE CLINIC YOKOHAMA)

- SURG** SURG mode for bone surgery
- PERIO** PERIO mode for maintenance
- ENDO** ENDO mode for root canal treatment



The tips are designed to preserve tissue, maintain cutting speed and easily cut into bone. Saw like teeth on an uneven blade edge enhances bone cutting. This also reduces clogging to maintain cutting speed.

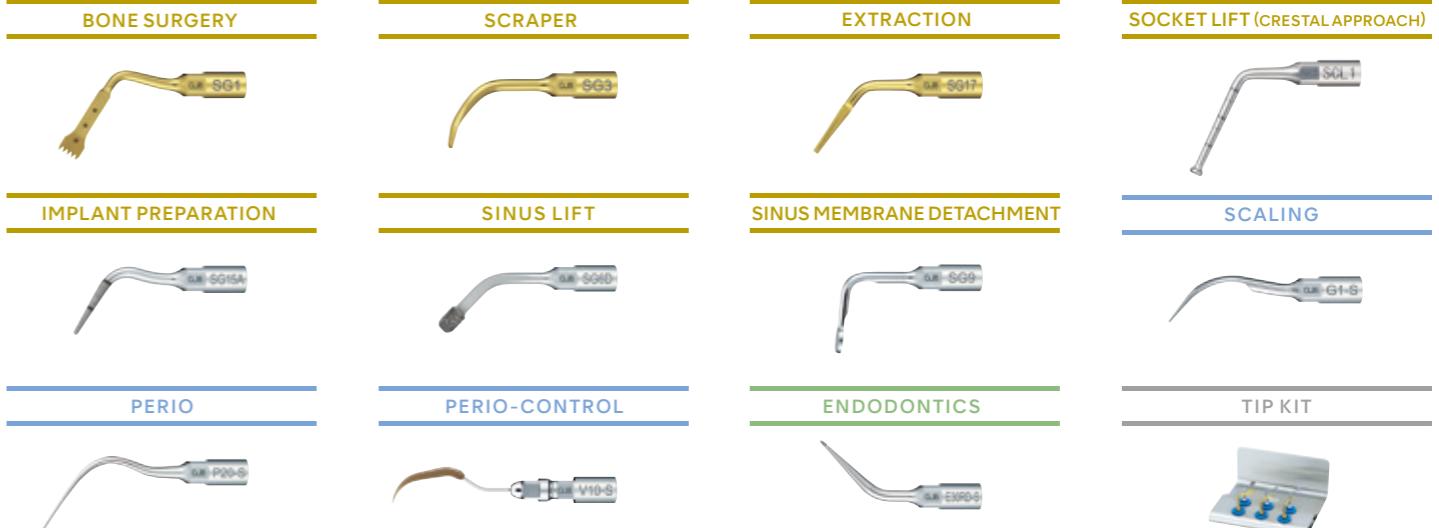

VarioSurg 3

EVOLVED POTENTIAL

Embodying the Pursuit of Perfection

Ultrasonic bone surgery systems are essential in modern surgical and implant treatment.

The powerful VarioSurg3 handpiece features a slim and lightweight body that ensures minimal hand fatigue and outstanding visibility. A variety of tips makes procedures easier and stress-free to help provide more precise and efficient treatment.





BONE SURGERY

BONE SURGERY

For chin bone harvests, notably for the mental foramen and mandibular ramus. For general bone surgery.

MODEL **SG1**

POWER LEVEL **SURG** 80%

MODEL **H-SG1**

POWER LEVEL **SURG** 150%

MODEL **SG1A**

POWER LEVEL **SURG** 80%



ORDER CODE


Z305101

- TiN coating • Dots mark 3, 6 and 9 mm from tip end
- Five teeth ; 0.6 mm thick



Unilateral anterior mandibular sampling
SG1, H-SG1, SG1A

HIGH POWER

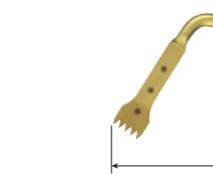

ORDER CODE

Z305151

- TiN coating • Dots mark 3, 6 and 9 mm from tip end
- Five teeth ; 0.6 mm thick



ORDER CODE

Z305138

- TiN coating • Dots mark 3, 6, 9, 12 and 15 mm from tip end
- Five teeth ; 0.6 mm thick

For clinical cases where movement is restricted around the end of the tip.

MODEL **SG8**

POWER LEVEL **SURG** 80%

MODEL **H-SG8**

POWER LEVEL **SURG** 150%

MODEL **SG8A**


POWER LEVEL **SURG** 80%



ORDER CODE

Z305109

- TiN coating • Dots mark 3, 6 and 9 mm from tip end
- Three teeth ; 0.6 mm thick

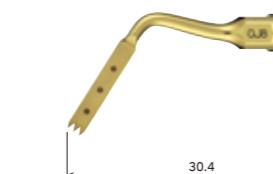

ORDER CODE

Z305155

- TiN coating • Dots mark 3, 6 and 9 mm from tip end
- Three teeth ; 0.6 mm thick



ORDER CODE

Z305139

- TiN coating • Dots mark 3, 6, 9, 12 and 15 mm from tip end
- Three teeth ; 0.6 mm thick

BONE SURGERY




For harvesting bone from the mandibular ramus.

MODEL **SG14R**

POWER LEVEL **SURG** 80%

MODEL **SG14L**

POWER LEVEL **SURG** 80%

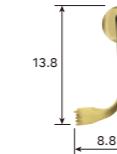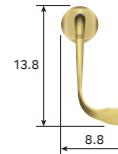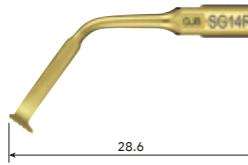

ORDER CODE

Z305122

- TiN coating • 90 degree right curved tip*
- Five teeth; 0.6 mm thick



Anterior mandibular harvesting
SG14R, SG14L

*The direction of the tip's curve is defined by the tip's anterior view.

BONE SURGERY

For harvesting bone from the mandibular ramus.



MODEL **H-SG8R**

POWER LEVEL **SURG** 150%

MODEL **H-SG8L**

POWER LEVEL **SURG** 150%

HIGH POWER

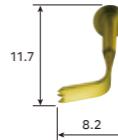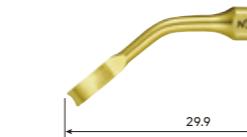

ORDER CODE

Z305156

- TiN coating • 90 degree right curved tip*
- Three teeth; 0.6 mm thick

ORDER CODE

Z305157

- TiN coating • 90 degree left curved tip*
- Three teeth; 0.6 mm thick


*The direction of the tip's curve is defined by the tip's anterior view.

BONE SURGERY

For excisions in areas with inclines or projections or which are otherwise hard to access with SG1 and other regular tips.

MODEL **SG2R**

POWER LEVEL SURG 80%

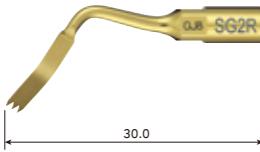

13.5
6.2

ORDER CODE
Z305102


- TiN coating • Right curved tip*
- Three teeth; 0.6 mm thick



Excision of bone torus
SG2R, SG2L

MODEL **SG2L**

POWER LEVEL SURG 80%

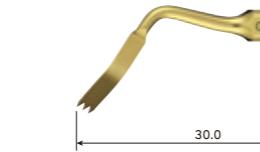

13.5
6.2


ORDER CODE
Z305103

- TiN coating • Left curved tip*
- Three teeth; 0.6 mm thick

MODEL **SG19**

POWER LEVEL SURG 80%

For cutting curved surfaces.

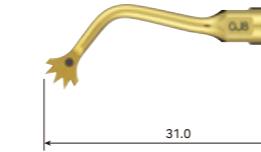

10.2

ORDER CODE
Z305135

- TiN coating • Dot mark 3 mm from tip end
- Five teeth; 0.8 mm thick



Mandibular harvesting
at approach stage
SG19

MODEL **SG30**

POWER LEVEL SURG 50%



12.9

ORDER CODE
Z305137


- TiN coating • Sharp edge • 0.5 mm thick



Alveolar crestal incision for
harvesting bone
(when narrowing bone cuts)
SG30

MODEL **SG58**

POWER LEVEL SURG 80%

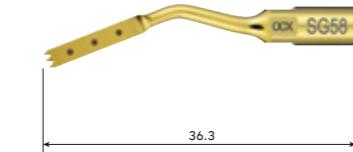

7.5

ORDER CODE
Z305141

- TiN coating • Dots mark 3, 6 and 9 mm from tip end
- Three teeth; 0.6 mm thick



The cutting part
is bone torus
SG58

MODEL **SG68**

POWER LEVEL SURG 80%



7.8

ORDER CODE
Z305143

- TiN coating • Marking every 2 and 4 mm from top of tip (Single side)
- Three teeth; 0.6 mm thick



Vertical osteotomy
for harvesting bone
SG68

* The direction of the tip's curve is defined by the tip's anterior view.

SCRAPER



SCRAPER

For flattening the alveolar crest or harvesting bone fragments.


MODEL
SG3

POWER LEVEL SURG 80%



ORDER CODE
Z305104


- TiN coating • Triple sided edge spatula tip
- Single-edged scraper

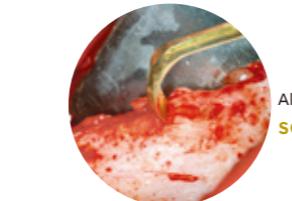

Alveolar crest flattening
SG3

MODEL
SG4

POWER LEVEL SURG 80%

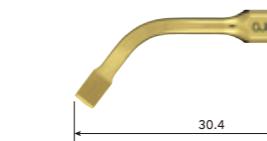

ORDER CODE
Z305105


- TiN coating • Edge spatula tip
- Double-edged scraper



Osteoplasties
SG5

MODEL
SG5

POWER LEVEL SURG 80%

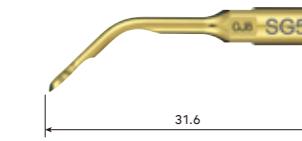

ORDER CODE
Z305106

- TiN coating • Rounded edge spatula tip
- Single-edged scraper





EXTRACTION

Ruptures the periodontal ligament to streamline suture removal. Also effective in extracting ankylosed teeth.

EXTRACTION

MODEL **SG17**

POWER LEVEL SURG 80%

MODEL **SG18R**


POWER LEVEL SURG 50%

MODEL **SG18L**


POWER LEVEL SURG 50%



13.0

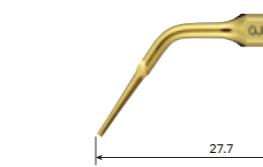

12.9

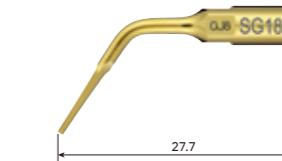

12.9

ORDER CODE

Z305132

• TiN coating • 0.7 mm thick

ORDER CODE

Z305133

• TiN coating • Right angled tip* • 0.7 mm thick


ORDER CODE

Z305134

• TiN coating • Left angled tip* • 0.7 mm thick



*The direction of the tip's angle is defined by the tip's anterior view.

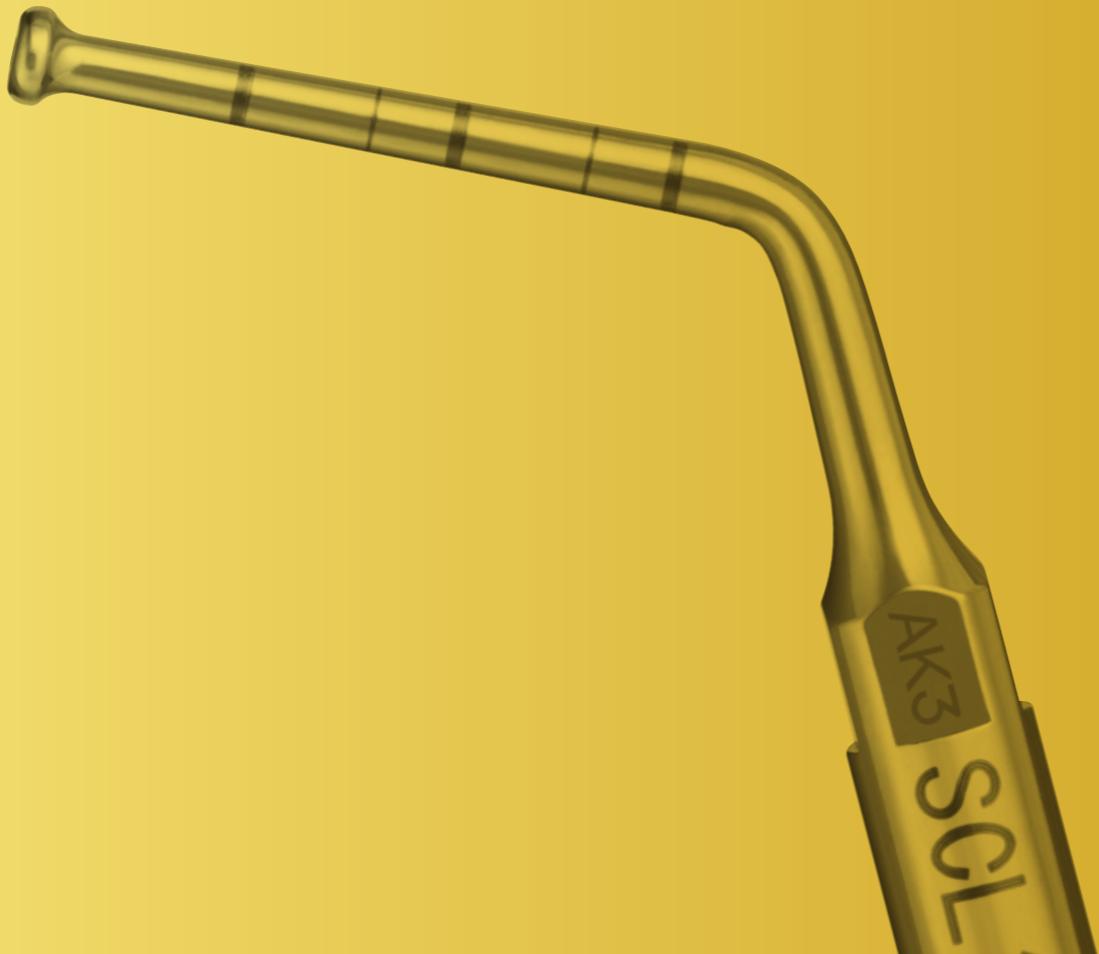


SOCKET LIFT
(CRESTAL APPROACH)

SOCKET LIFT (CRESTAL APPROACH) Socket lift procedures.

The socket lift tip enhances the safety of procedures, being less invasive and enabling sinus lifts with short chair times.

Osteotomy with ultrasonic vibration reduces the risk of damage to the Schneiderian membrane.



Eliminating hammering taps through the osteotome technique reduces patient discomfort and potential dizziness.

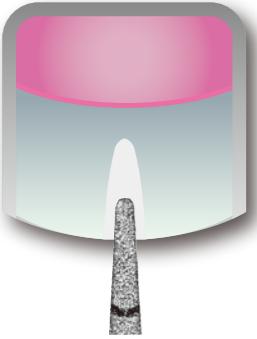

A starting point is added to trigger bone formation.

Bone formation to prepare implant site.

Implant site expansion and formation.

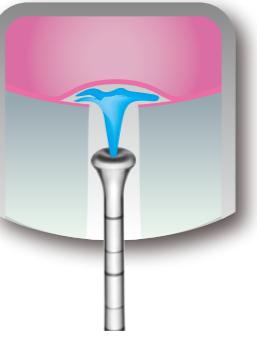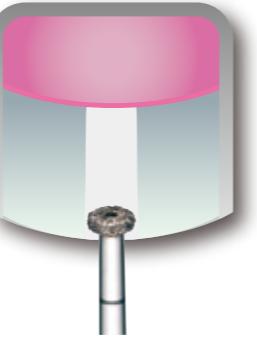


IMPLANT PREPARATION

For preparation of implant sites, adding starting points, and other steps.

MODEL **SG15A**

POWER LEVEL SURG 50%

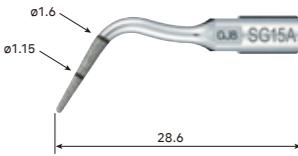

ORDER CODE
Z305124


- Diamond coating • Tip-end diameter; 0.7 mm
- Markings every 5 mm and 10 mm from tip end



Implant preparation
SG15A, SG15B

MODEL **SG15B**

POWER LEVEL SURG 50%

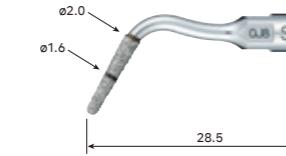


ORDER CODE
Z305125

- Diamond coating • Tip-end diameter; 1.3 mm
- Markings every 5 mm and 10 mm from tip end

For preparation of implant sites, flattening bone surfaces, and other steps.

MODEL **SG15C**

POWER LEVEL SURG 50%

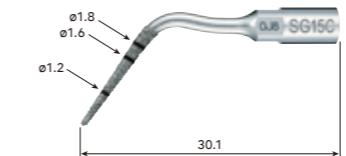

ORDER CODE
Z305126

- Diamond coating • Tip-end diameter; 0.9 mm
- Markings every 5 mm, 10 mm and 12 mm from tip end



Implant preparation
SG15C, SG15D

MODEL **SG15D**

POWER LEVEL SURG 50%



ORDER CODE
Z305127

- Diamond coating • Tip-end diameter; 1.3 mm
- Marking every 5 mm, 10 mm and 12 mm from tip end



IMPLANT PREPARATION


For expanding and forming implant preparation site when approaching from the alveolar crest.

MODEL **SG16A**

POWER LEVEL SURG 50%

MODEL **SG16B**

POWER LEVEL SURG 50%

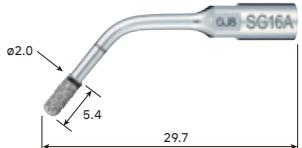

ORDER CODE

Z305128

- Diamond coating • Diameter; 2.0 mm
- Markings every 5 mm and 10 mm from tip end



Implant preparation
SG16A



Implant preparation
SG16B



For expanding and forming implant preparation site when approaching from the alveolar crest.


MODEL **SCL2D**

POWER LEVEL SURG 50%

MODEL **SCL4D**

POWER LEVEL SURG 80%

INTERNAL IRRIGATION

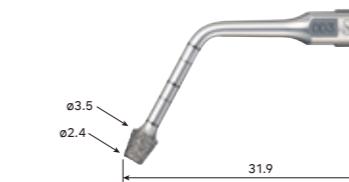

ORDER CODE

Z305182

- Diamond coating



Implant preparation site depth can be measured using markings on the tip.



Formation of implant preparation site
SCL2D, SCL4D


SOCKET LIFT (CRESTAL APPROACH)

For procedures such as osteotomy of maxillary sinus approaching from the alveolar crest and implant preparation site adjustments.

MODEL **SCL1D**

POWER LEVEL SURG 80%

INTERNAL IRRIGATION

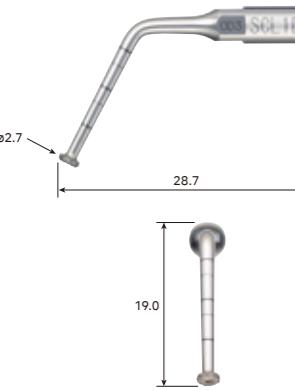

ORDER CODE
Z305171

• Diamond coating




Implant preparation site depth can be measured using markings on the tip.

MODEL **SCL3D**

POWER LEVEL SURG 80%

INTERNAL IRRIGATION

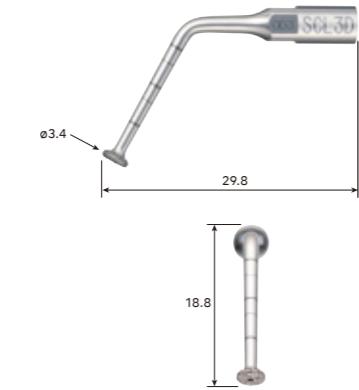

ORDER CODE
Z305173

• Diamond coating



Cutting the cavity floor of implant preparation site
SCL1D, SCL3D, SCL5D

MODEL **SCL5D**

POWER LEVEL SURG 80%

INTERNAL IRRIGATION



ORDER CODE
Z305175

• Diamond coating

Elevating the sinus membrane with internal irrigation approaching from alveolar crest.

MODEL **SCL1**

POWER LEVEL SURG 50%

INTERNAL IRRIGATION



ORDER CODE
Z305170

• Diamond coating



Implant preparation site depth can be measured using markings on the tip.

MODEL **SCL3**

POWER LEVEL SURG 50%

INTERNAL IRRIGATION



ORDER CODE
Z305172

• Diamond coating



Before the mucous membrane of maxillary sinus is elevated
SCL1, SCL3, SCL5

MODEL **SCL5**

POWER LEVEL SURG 50%

INTERNAL IRRIGATION



ORDER CODE
Z305174

• Diamond coating

SOCKET LIFT (CRESTAL APPROACH)



SINUS LIFT

For bone fenestration and bone formations.

MODEL **SG6D**

POWER LEVEL SURG 80%



ORDER CODE
Z305107


• Diamond coating



Avulsion of wisdom teeth
and impacted teeth
SG6D

MODEL **SG7D**

POWER LEVEL SURG 50%

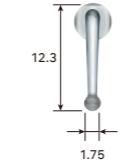

ORDER CODE
Z305108

• Diamond coating




Lateral maxillary posterior
bone augmentation
SG7D

For detachment and elevation of the maxillary sinus membrane.

SINUS MEMBRANE DETACHMENT

MODEL **SG9**

POWER LEVEL SURG 50%

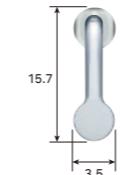

ORDER CODE
Z305110


• Flat circular convex elevator • Angled at 90°



Sinus membrane detachment
SG9, SG10

MODEL **SG10**

POWER LEVEL SURG 50%

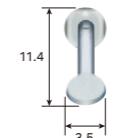


ORDER CODE
Z305111

• Flat circular convex elevator • Angled at 135°

MODEL **SG11**

POWER LEVEL SURG 50%

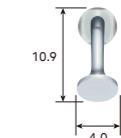


ORDER CODE
Z305112

• Cone compressor



Sinus membrane elevation
SG11

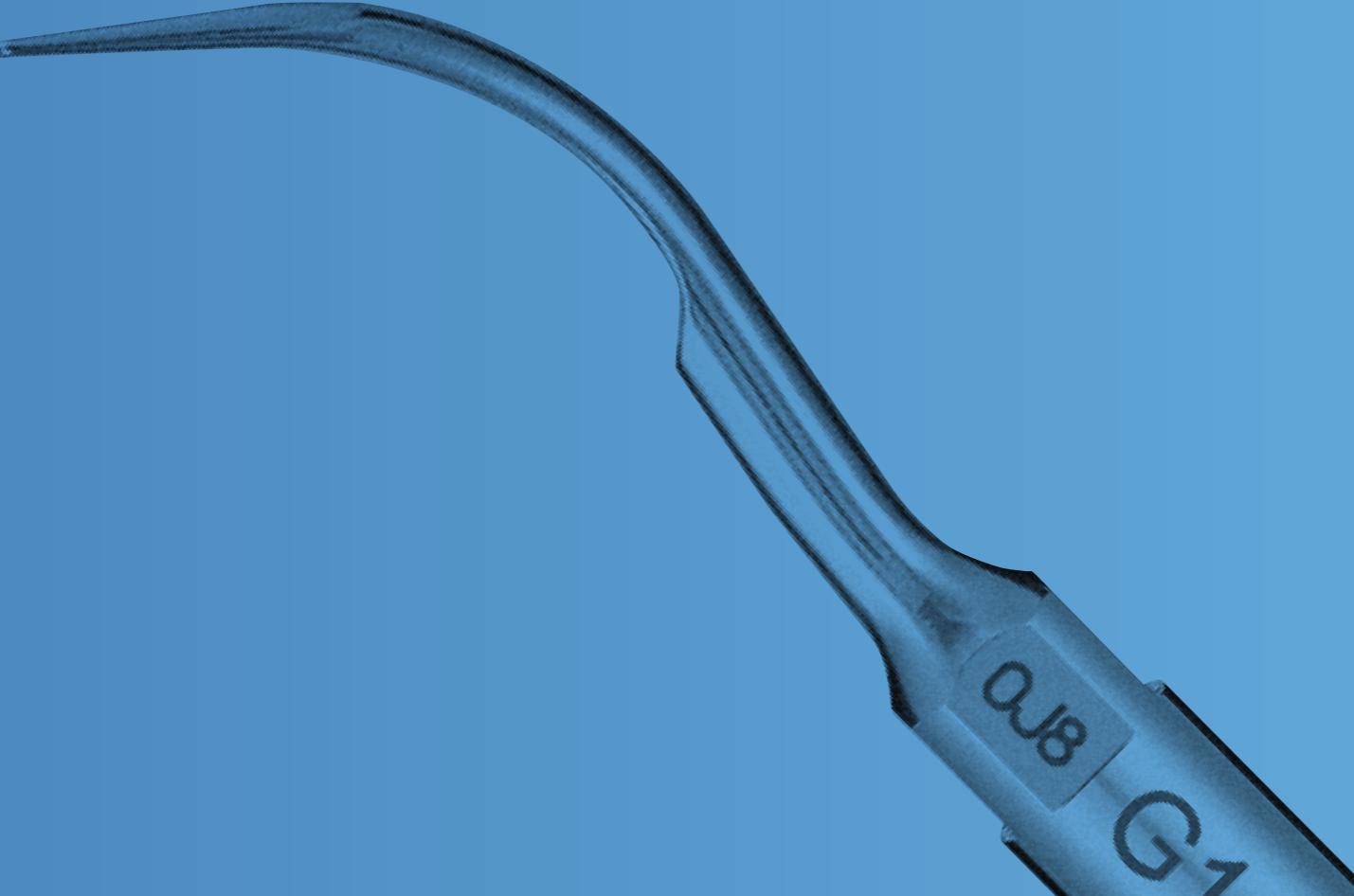

SCALING

SCALING

Removal of supra gingival tartar, tartar narrow interdental space, marginal gingival calculus.

MODEL **G1-S**

POWER LEVEL **SURG** 50%



4.5
7.5

ORDER CODE
Z305113



Single rooted / Multi rooted
G1-S

Removal of gingival biofilm. **PERIO**

MODEL **P20-S**

POWER LEVEL **PERIO** 100%



4.5
12.3

ORDER CODE
Z305114



Single rooted / Multi rooted
P20-S

MODEL **P25R-S**

POWER LEVEL **PERIO** 100%



6.0
7.6

ORDER CODE
Z305115

* Right curved type*



Single rooted / Multi rooted
P25R-S, P25L-S

MODEL **P25L-S**

POWER LEVEL **PERIO** 100%



6.0
7.6

ORDER CODE
Z305116

* Left curved type*

PERIO-CONTROL

For maintaining the upper structure of implant-supported dental prosthesis.



MODEL **V10-S**

POWER LEVEL **PERIO 100%**

MODEL **V-P10**



For the maintenance of prostheses and implants. Produced from special material to allow safe debridement without causing damage to metals and implants.

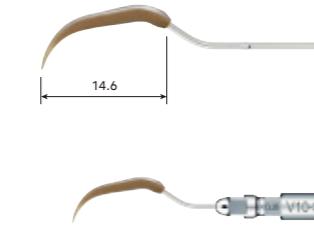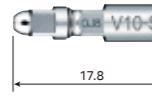

7.9

ORDER CODE

Z305117

- Includes E-Tip replacement wrench
- Plastic Tip is not included



V-P10

PERIO-CONTROL


For maintaining the upper structure of implant-supported dental prosthesis.

MODEL **V-P11R**


MODEL **V-P11L**

MODEL **V-P12**


Double bent tips. This tip is extra long to easily access difficult to reach areas and to avoid interference with neighbouring teeth or contours.

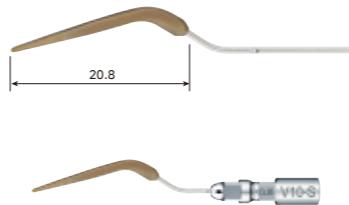

4.9

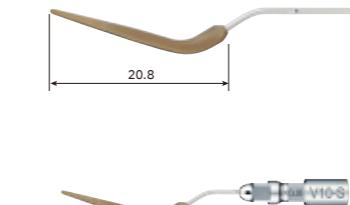

4.9

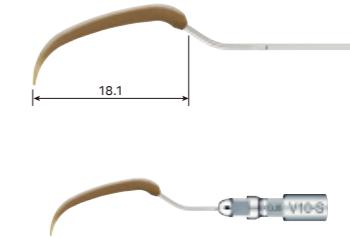

9.6

ORDER CODE

Y1002165

- Pack of 3 • Right curved type*
- V10-S holder is not included



V-P11R, V-P11L

ORDER CODE


Y1002166

- Pack of 3 • Left curved type*
- V10-S holder is not included



V-P12

ENDODONTICS

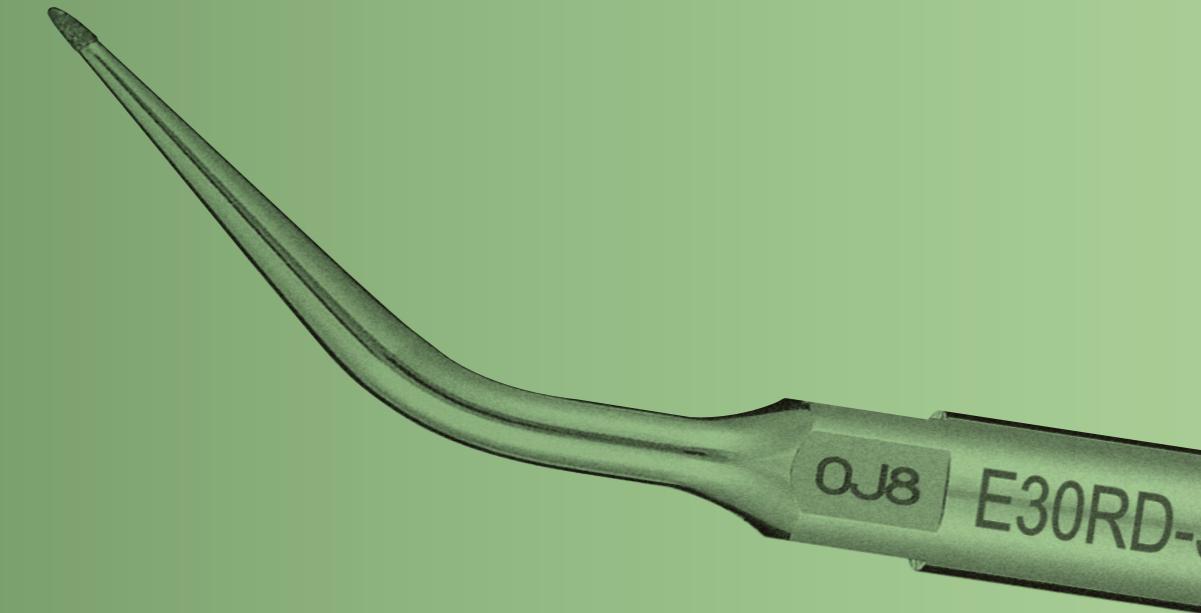

The microscopic diamond coated tip allows effective removal of dentin at the apex.

MODEL **E30RD-S**

POWER LEVEL
ENDO 50%

MODEL **E30LD-S**


POWER LEVEL
ENDO 50%

MODEL **E31D-S**

POWER LEVEL
ENDO 50%

MODEL **E32D-S**

POWER LEVEL
ENDO 50%

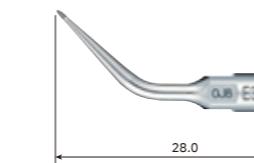


ORDER CODE
Z305118

- Diamond coating
- For posterior teeth (right angled*)





ORDER CODE
Z305119

- Diamond coating
- For posterior teeth (left angled*)

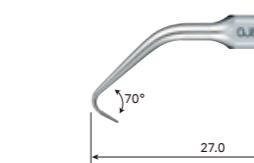


ORDER CODE
Z305120

- Diamond coating
- For anterior and posterior teeth (70°)

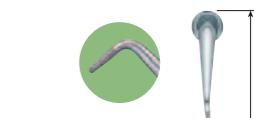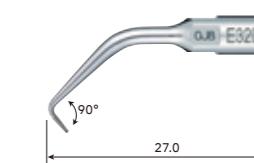


ORDER CODE
Z305121

- Diamond coating
- For anterior teeth (90°)

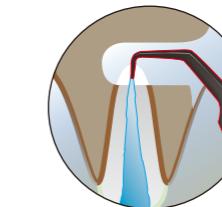

* The direction of the tip's angle is defined by the tip's anterior view.

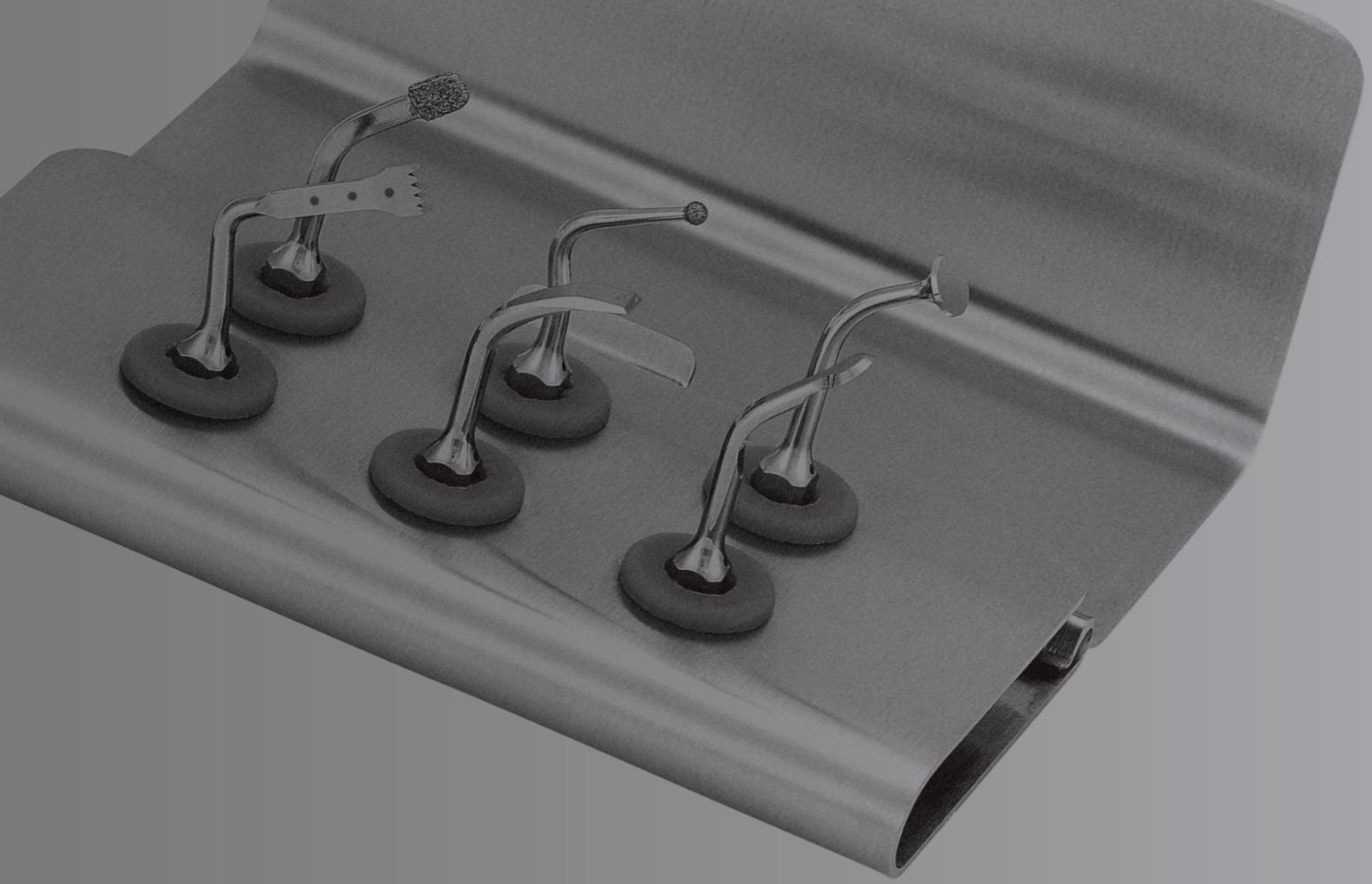

TIP KIT

TIP KIT

MODEL Basic H-S Kit

Contents

- H-SG1, SG3, SG5, SG6D, SG7D, SG11
- Tip holder

BONE SURGERY
SINUS LIFT
SCRAPER
SINUS MEMBRANE DETACHMENT



ORDER CODE
Y1002775

MODEL Sinus Lift Kit

Contents

- SG1, SG3, SG6D, SG9, SG10, SG11
- Tip holder

SINUS LIFT



ORDER CODE
Y900689

MODEL Bone Cut Kit

Contents

- SG1, SG2R, SG4, SG2L, SG6D
- Tip holder

BONE SURGERY



ORDER CODE
Y900688

MODEL Endo-S Kit

Contents

- G1-S, E30RD-S, E30LD-S, E31D-S, E32D-S
- Tip holder

ENDODONTICS



ORDER CODE
Y900691

MODEL Implant Preparation Kit

Contents

- SG15A, SG15B, SG16A, SG16B
- Tip holder

IMPLANT PREPARATION



ORDER CODE
Y900774

MODEL Socket Lift (CRESTAL APPROACH) Kit for Wide Size Implant

Contents

- SCL2D, SCL3D, SCL4D, SCL3
- VS Tip Wrench • Tip holder • Manual

SOCKET LIFT(CRESTALAPPROACH)



ORDER CODE
Y1002842

MODEL Socket Lift (CRESTAL APPROACH) Kit for Regular Size Implant

Contents

- SCL1D, SCL2D, SCL1
- VS Tip Wrench • Tip holder • Manual

SOCKET LIFT(CRESTALAPPROACH)



ORDER CODE
Y1002841

VarioSurg 3

Advantages

- Super slim handpiece with LED
- Extensive tip range for a wide range of applications
- Link 2 system with Surgic Pro2 (implant motor)



Complete Set

Optic	MODEL	ORDER CODE
●	VarioSurg3 (230V)	Y1002726

Contents

- Control Unit • LED Handpiece with 2m cord • FC-78 Foot Control
- Sterilization Cassette • Handpiece Stand
- Irrigation tube (5 pcs.) and other accessories
- Basic H-S Kit (H-SG1, SG3, SG5, SG6D, SG7D, SG11 and Tip holder)

Complete Set without Foot Control

Optic	MODEL	ORDER CODE
●	VarioSurg3 (230V) Non FT	Y1002248

Contents

- Control Unit • LED Handpiece with 2m cord • Sterilization Cassette
- Handpiece Stand • Irrigation tube (5 pcs.) and other accessories
- Basic H-S Kit (H-SG1, SG3, SG5, SG6D, SG7D, SG11 and Tip holder)

Specifications (Control Unit)

- Frequency : 28-32 kHz
- Power Supply : 230V 50/60 Hz
- Irrigation Flow Rate : 10-75 mL / min
- Programs : SURG x 5, ENDO x 2, PERIO x 2
- Dimensions : W 265 x D 220 x H 100 mm

NAKANISHI INC. www.nsk-dental.com
700 Shimohinata, Kanuma
Tochigi 322-8666, Japan

NSK Europe GmbH www.nsk-europe.de
Elly-Beinhorn-Strasse 8
65760 Eschborn, Germany

NSK United Kingdom Ltd. www.nsk-uk.com
Office 4, Gateway 1000, Arlington Business Park
Whittle Way, Stevenage, SG1 2FP, UK

NSK Dental Nordic www.nsk-nordic.com
Furstenbergsgatan 4
416 64 Gothenburg, Sweden

Specifications are subject to change without notice.